

AURIDENT POLISHING KIT

The Aurident Polishing Kit consists of selected Brasseler components and a specially formulated non-toxic alumina based polishing compound. This system is designed to produce an outstanding shine on noble metal alloys. It is highly effective on difficult to polish surfaces encountered with the softer, high gold content yellow ceramic alloys.

The kit contains four diamond impregnated polishing wheels (0401-220, GGG, GYM, and CPF), three diamond impregnated polishing tapers (0041-047, 0141-047, BRY-0141), one carbide cutter for trimming and de-burring (138EF) and one polishing brush (16222HP). Each component is color coded to indicate the degree of abrasiveness with blue being the coarsest and yellow, brown and green being progressively finer. The polishing tapers are used on surfaces that are not accessible with the wheels. The coarseness of the tapers follows the order: Brown for pre-polishing, Green for fine polishing, and Green with Yellow Band for high shine polishing. For best results apply light pressure.

To provide a high luster restoration, the Aurident Polishing Compound is used. Manually polish the metal restoration's surfaces using the Brasseler wheels and cones at medium to high speed and light pressure. Apply a small amount of Aurident Polishing Compound to the surface and use the brush to polish at medium to high speed with light pressure. Thoroughly clean and prepare the finished restoration using distilled water or acetone in a sonic cleaner for 1 minute and dry before preparing the final restoration.